INSTRUCTIONS FOR USE

Figulla[®] Flex II PFO Occluder

English	3
Deutsch	16
Español	30
French	44
Italiano.....	57
Polski.....	70
Português.....	85
Turkish	98

TABLE OF CONTENTS

1. Product description	3
2. Indication for Use and Area of Application	4
3. Contraindications.....	4
4. Warnings	5
5. Preventative measures	6
5.1 Handling	6
5.2 Device Size	7
6. Adverse events.....	7
7. Magnetic Resonance Imaging (MRI) safety information	7
8. Implantation Procedure.....	8
8.1 Measurement of Defect and Device Sizing	8
8.2 Procedural Method of Implantation	8
8.3 Notes for Troubleshooting	12
8.4 Report of Serious Adverse Events.....	12
8.5 Recommended Follow-up Examinations.....	12
8.6 Recommended Medication after Implantation	13
9. Device Sizes and Recommended delivery systems	13
10. Storage conditions	14
11. Disposal	14
12. Labelling.....	15

1. PRODUCT DESCRIPTION

The Figulla® Flex II Patent Foramen Ovale (PFO) Occluder is a percutaneous, transcatheter, atrial septal defect closure device, which consists of a Nitinol wire mesh. It consists out of two retention discs, connected through a thin flexible ridge. The design properties allow the device to be pulled into a delivery sheath and reshape once it is released. Once the device is correctly positioned and firmly attached on both sides of the septum, it conforms to the septal wall and closes a present defect. Two very thin polyester patches sewn in the left atrial (LA) and right atrial (RA) disc ensure the instant stop of blood flow through the defect as well as the optimal tissue growth.

The Figulla® Flex II PFO Occluder is packed together with its compatible pusher as shown in Figure 1.

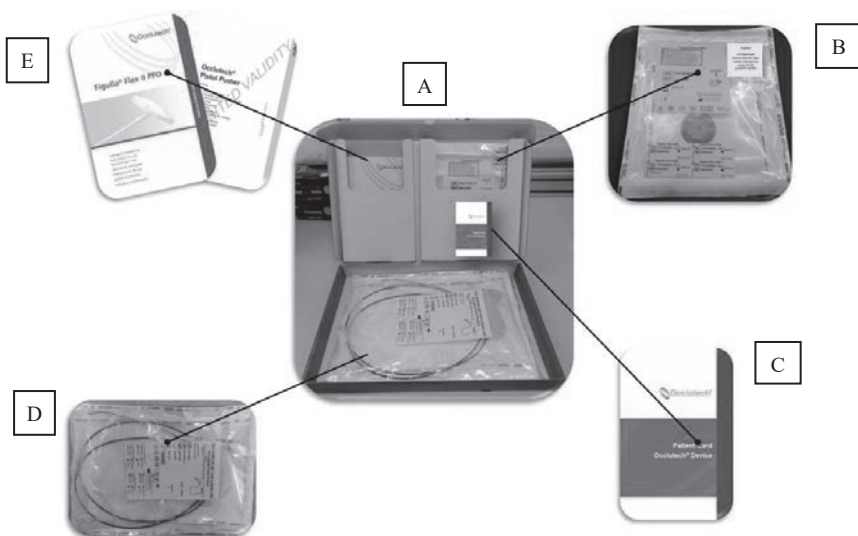
Figure 1: (A) - Product box as delivered to the user; (B) - Figulla® Flex II PFO Occluder; (C) – Patient Card; (D) - Pusher (for example: Occlutech® Pistol Pusher); (E) – Instructions for Use (one for the Figulla® Flex II PFO Occluder and one for the Pusher system).

2. INDICATION FOR USE AND AREA OF APPLICATION

The Figulla® Flex II PFO Occluder is a medical device for transcatheter closure of Patent Foramen Ovale (PFO). It is intended for patients with substantiated right-to-left-shunt in the contrast agent echocardiography under Valsalva maneuver and presence of a “cryptogenic”, paradoxical thromboembolism.

The Figulla® Flex II PFO Occluder D is available in four different sizes (Section 9, Table 3: Device Sizes and Recommended Delivery Systems) suitable for each diameter of PFO defect up to 18 mm including tunnel-shaped PFOs. Such an anatomy of a PFO is ascertained by means of Sizing-Balloon aided X-ray evaluation and/or echocardiographic imaging (TEE and/or TTE).

For the size 23/25, two types of PFO Occluder are available: Figulla® Flex II PFO Occluder S (Single Disc) and Figulla® Flex II PFO Occluder D (Double Disc). The Figulla® Flex II PFO Occluder S offers the advantage of minimization of the left atrial material. However, it is not recommended in case of tunnel-shaped defects.

3. CONTRAINDICATIONS

The Figulla® Flex II PFO Occluder is contraindicated for in the following cases:

- Acute infection
- Adequate oral anti-coagulation therapy / platelet inhibition is not possible post-intervention
- Arrhythmia
- Atrial tumor
- Atrial thrombus
- Eisenmenger-syndrome
- Known coagulation disorder
- Intolerance to contrast agents

- Allergies to nickel and/or titanium and/or nickel and/or titanium-related materials
- Recent pelvic venous thrombosis
- Recent myocardial infarction or bypass surgery within the last 30 days
- Patients whose size or condition would cause the patient to be a poor candidate choice for cardiac catheterization (e.g. too small for echocardiography imaging probe, catheter size, vasculature size, active infection, body weight < 8 kg)

4. WARNINGS

The Figulla® Flex II PFO Occluder must be implanted exclusively by physicians trained in its use and who are experienced with interventional transcatheter techniques.

- Physicians who implant the Figulla® Flex II PFO Occluder must be able to recognize procedure-associated emergency situations, and be able to assess and manage such situations appropriately. On-site cardiac surgical support with corresponding personnel must be available.
- Patients allergic to nickel and/or titanium and/or nickel/titanium-based materials may suffer an allergic reaction to this device. Certain allergic reactions can be serious; patients shall be instructed to notify their physicians immediately if they suspect they are experiencing an allergic reaction such as difficulty in breathing, or inflammation of the face or throat.
- The use of improperly size-matched devices could seriously affect hemodynamics and optimal results. Before using this device, physicians shall carefully review the hemodynamic parameters as well as sizing information printed on the labels of the Figulla® Flex II PFO Occluder and its accessories (pusher and Occlutech® Delivery Set, ODS). Physicians shall also review the sizing and compatibility chart in Section 9 (Table 3: Device Sizes and Recommended Delivery Systems) of this Instructions for Use (IFU) before starting a procedure. Careful consideration shall be given to ensure accurate size matching of accessories (pusher and ODS) with the corresponding device (i.e. review of device labels and color coding).
- Before using the Figulla® Flex II PFO Occluder, the physician shall carefully review Section 8 (Implantation Procedure) including relevant instructions therein on connecting the Figulla® Flex II PFO Occluder to the pusher. If loading is done improperly and a secure connection between the Figulla® Flex II PFO Occluder and pusher has not been verified as described, unnoticed disconnection of the device from its pusher may occur and the Figulla® Flex II PFO Occluder may embolize which can present a life-threatening situation to the patient.
- The Figulla® Flex II PFO Occluder shall not be used with delivery systems other than those recommended in Section 9 (Table 3: Device Sizes and Recommended Delivery Systems) of this IFU.
- After deployment and release of the Figulla® Flex II PFO Occluder, complications such as device dislocations or embolizations may occur as result of erroneous positioning or sizing of the device. These complications can present a life-threatening situation to the patient.
- Embolized Figulla® Flex II PFO Occluder must be removed. An emergency kit for the retrieval of the Figulla® Flex II PFO Occluder must be available in the catheterization laboratory during the procedure.
- The Figulla® Flex II PFO Occluder shall only be released from its pusher after the physician has ascertained that the device is positioned correctly. This should be done by performing fluoroscopy and/or Transesophageal (TEE), Transthoracic (TTE) or Intracardiac Echocardiography (ICE) to visualize the Figulla® Flex II PFO Occluder and to confirm that the device is positioned properly.
- Do not release the device from the pusher if the Figulla® Flex II PFO Occluder does not conform to its original configuration; if the device position is unstable, or if the device interferes with any adjacent cardiac structures such as the superior vena cava (SVC), pulmonary vein (PV), mitral valve (MV), coronary sinus (CS) or aorta (AO). In this case a

correction of the position is required. In this case, please follow the guidance in Section 8.3: Notes for Troubleshooting, “incorrectly positioned device”. If it is not possible to improve the position of the Figulla® Flex II PFO Occluder, the device should be withdrawn into the delivery sheath, removed and discarded.

- If it has not been released from the pusher, the Figulla® Flex II PFO Occluder can be retrieved or re-positioned using the recommended delivery system. When retrieving the device, the Figulla® Flex II PFO Occluder shall not be moved through any intracardiac structure as this may cause damage to intracardiac structures. This can present a life-threatening situation to the patient.
- The Figulla® Flex II PFO Occluder is to be used exclusively in accordance with these Instructions for Use and its implantation is to be carried out as described in this IFU.
- The physician shall inspect all packaging and labels of all devices before opening and follow the instructions. If the product box or sterile packaging are damaged in any manner, the Figulla® Flex II PFO Occluder shall be considered as unsterile and should not be used.
- The physician shall not use this device or any of its components if a seal appears to be or is broken (contents may not be sterile); if the label appears marked with text or symbols other than those on the label shown in this IFU or if the label is illegible, inappropriate, or absent.
- The physician shall not use this device or any of its components after the “use by” (expiration) date.

The Figulla® Flex II PFO Occluder is intended for single use only, this medical device is not suitable for re-sterilization. As soon as the sterile seal of the device is broken, it is contaminated. Re-use or re-sterilization may compromise the structural integrity of the devices, lead to device failure, and result in patient injury, illness or death.

- If, after inspection by the physician, the Figulla® Flex II PFO Occluder appears damaged or does not appear to function properly, the device is not suitable for implantation and should be discarded.
- Patients should be advised to avoid strenuous physical activity for a period of at least two weeks after device implantation.
- Patients with a rim of < 9 mm from the aortic root or SVC may have a higher risk of erosion and if closed using a device, they will require closer follow up.
- Patients with rim sizes < 5 mm to the coronary sinus, inferior vena cava rim, an atrioventricular valve, or the right upper pulmonary vein may have a higher risk of device embolization and it is best to avoid doing such cases.

5. PREVENTATIVE MEASURES

5.1 Handling

Prior to introducing the Figulla® Flex II PFO Occluder in the patient, it is recommended to check whether the device reverts to its original shape once it has been pulled in and pushed out of the loader while immersed in a sterile physiological NaCl solution. If the Figulla® Flex II PFO Occluder does not revert to its intended shape, it is not suitable for implantation and shall be discarded (see details in Section 8: Implantation Procedure). The physician shall inspect the device prior to its use in patient and check the following.

- Pusher size is correct (match color codes);
- The Figulla® Flex II PFO Occluder can be properly connected to the pusher;
- The Figulla® Flex II PFO Occluder can be released from the pusher by operating the release mechanism;
- The Figulla® Flex II PFO Occluder is firmly attached to the pusher by gently pulling the device a few times;

- The Figulla® Flex II PFO Occluder reverts to its original shape once it has been pulled in and pushed out of the loader while immersed in a sterile physiological NaCl-solution.

In case of any kind of inconsistency, the physician shall select another device.

5.2 Device Size

A suitable Figulla® Flex II PFO Occluder can be determined based on measured PFO size (see Section 8.1: Measurement of Defect and Device Sizes) and given device-related sizing information in Section 9 (see Table 3: Device Sizes and Recommended Delivery Systems) identifying a device which can match this defined PFO defect.

6. ADVERSE EVENTS

- Air embolism
- Allergic reactions
- Anesthesia reactions
- Apnea
- Arrhythmias
- AV-Fistula
- Bleeding (hemorrhage) requiring treatment
- Cardiac / vascular perforation
- Cardiac tamponade
- Death
- Embolization (peri & post procedural)
- Esophageal injury
- Femoral access complication
- Fever
- Hematoma
- Hypertension or Hypotension
- (Immediate) surgical interventions
- Infections, including endocarditis
- Pericardial effusions
- Pericarditis
- Post-pericardiotomy syndrome
- Pseudo aneurysm
- Pulmonary edema
- Seizure
- Stroke
- Tissue erosion
- Thrombus formation on device
- Thrombosis
- Valvular regurgitation

7. MAGNETIC RESONANCE IMAGING (MRI) SAFETY INFORMATION

Non-clinical testing and MRI simulations were performed to evaluate the entire family of the Figulla® Flex II PFO Occluder.

Non-clinical testing demonstrated that the entire family of this implant is MR Conditional. A patient with an implant from this family can be scanned safely in an MR system under the following conditions:

- Static magnetic field of 1.5-Tesla and 3-Tesla, only
- Maximum spatial gradient magnetic field of 4,000-gauss/cm (40-T/m)
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 2-W / kg for 15 minutes of scanning (i.e., per pulse sequence) in the Normal Operating Mode

Under the scan conditions defined, the Figulla® Flex II PFO Occluder is expected to produce a maximum temperature rise of 2.3 °C after 15-minutes of continuous scanning (i.e., per pulse sequence).

Artifact Information

In non-clinical testing, the image artifact caused by the Figulla® Flex II PFO Occluder extends approximately 10-mm from this device when imaged with a gradient echo pulse sequence and a 3-Tesla MR system.

8. IMPLANTATION PROCEDURE

The Figulla® Flex II PFO Occluder is intended for transcatheter delivery and techniques applied by physicians can vary. To minimize the risk of adverse events occurring, the following manufacturer's recommendations must be reviewed before implantation of this device.

8.1 Measurement of Defect and Device Sizing

The PFO defect should be sized by means of echocardiographic imaging methods or alternatively by a sizing-balloon and using the “stop-flow method”. For best results, the sizing balloon should be inflated until the right-to-left shunt ceases (“stop-flow”). The minimum diameter that achieves a stop-flow should be measured using TEE, TTE or ICE. If a balloon waist is not clearly visible, radiographic measurements may also be used to confirm the PFO defect diameter.

Note: To mitigate the risk of erosion, physicians must accurately evaluate the atrial septal defect. Implantation of the Figulla® Flex II PFO Occluder in patients with deficient retro-aortic rim less than 9 mm (“naked aorta”) or a deficient superior rim significantly increases the risk of erosion.

Based on the measured PFO defect size, a suitable device size can be determined based on the given device-related sizing information in Section 9 (see Table 3: Device Sizes and Recommended Delivery Systems).

8.2 Procedural Method of Implantation

Required Accessories (not included in Figulla® Flex II PFO Occluder Set)

- Occlutech® Delivery Set (ODS) containing:
- delivery sheath and dilator, loader and Y-hemostatic valve Guidewire of sufficient stiffness, e.g. Occlutech Stiff Guidewire Art. No. 52GWR01
- Multi-purpose catheter e.g. 5 F
- Luer lock stopcock
- appropriate accessories to manage potential adverse events

Recommended Imaging Tools

- X-ray fluoroscopy
- TTE, TEE or ICE

Recommended Medication

- Anti-platelet / anti-coagulant therapy is indicated regarding patient conditions: On the day before the procedure, it is recommended to pre-medicate the patient with acetylsalicylic acid (100 to 500 mg) and Clopidogrel (300 mg) loading doses.
- On the day of the procedure, standard endocarditis prophylaxis (e.g. a second or later generation Cephalosporin) is administered.
- The patient should be heparinized during the entire implantation procedure, according to standard hospital procedures. An Activated Clotting time (ACT) of > 200 seconds is recommended.

Implantation

Note: The use of TTE, TEE or ICE is required before, during, and after implantation. If TEE is used, the patient's esophageal anatomy must be adequate for the placement and manipulation of the TEE probe.



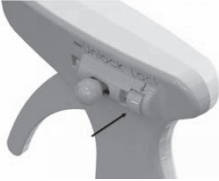
1. Access the vein circulation and perform a Standard Right Heart Catheterization.
2. A 0.035" guidewire is introduced and will be advanced through the atrial communication to be positioned in one of the pulmonary veins. A 5-French-multipurpose-catheter is maneuvered over the guidewire through the PFO into the left atrium.
3. Sizing the PFO defect by means of echocardiographic measurements and in addition by a sizing-balloon. Verify the flow status over the defect using doppler ultrasound. ("stop-flow method"). X-ray evaluation on the waist of the sizing balloon by using the reference markers equals defect size. Remove the sizing balloon.
4. Based on PFO defect as well as noting further intracardiac structures and device-related sizing information given in Section 9 (see Table 3: Devices Sizes and Recommended Delivery Systems), select an appropriate Figulla® Flex II PFO Occluder and a size-matched Delivery Set (e.g. Occlutech® Delivery Set [ODS]).
5. Prepare the delivery sheath as described in its IFU, connect a stopcock on the side arm of the proximal arm of the y-connector and introduce it into the right atrium under echo or x-ray guidance.

Note: Do not abruptly remove the dilator from the left atrium as the risk for causing an air embolism can occur.

6. Remove the pusher and the Figulla® Flex II PFO Occluder from sterile packaging followed checking of device integrity and pusher functionality.
7. Connect the Y-hemostasis valve to the loader (both provided with the ODS). Insert the distal end (jaw-container) of the pusher through the Y-hemostasis valve into the loader; please see ODS IFU for detailed device components and descriptions).
8. The Figulla® Flex II PFO Occluder should be placed in a bowl filled with a sterile physiological NaCl solution. Follow the instructions given in Table 1 (step 8a to 8e), to connect the Figulla® Flex II PFO Occluder to the pusher:

Table 1: Device-Pusher-Connection

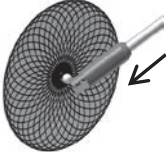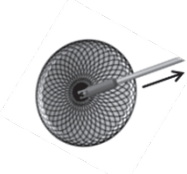Steps	Flex – Pusher II (FP II)	Occlutech® Pistol Pusher (OPP)
<p>8a. Open the jaws of the applied pusher (please see pusher IFU for detailed device components and descriptions).</p>	<p>Turn the FP II lock mechanism (shown in red) to the 'unlock' position.</p>  <p>Push the handle at the proximal end of the FP II forward into distal direction (opens jaws).</p>	<p>Please be sure that 'unlock button' is in the correct unlock position.</p>  <p>Pull the trigger located in the handle of OPP to control unlock position of handle and to release the jaws while pusher is in a straight position.</p>

Steps	Flex – Pusher II (FP II)	Occlutech® Pistol Pusher (OPP)
		
<p>8b. Connect the open pusher jaws to the ball-connector located on the Figulla® Flex II PFO Occluder</p>		
<p>8c. Retracting the jaws to connect the Figulla® Flex II PFO Occluder</p>	<p>Release the FP II handle.</p>	<p>Release the trigger mechanism of the OPP.</p>
<p>8d. Securing the device-pusher connection while the pusher is in a straight position.</p> <p>Note: A secure device-pusher connection is required to prevent any unintentional release of the device from the pusher.</p>	<p>Turn the FP II lock mechanism (shown in red) to its ‘locked’ position.</p> 	<p>Push the lock button of the OPP to its ‘locked’ position.</p> 
<p>8e. Figulla® Flex II PFO Occluder / pusher assembly</p>		

Note: The Figulla® Flex II PFO Occluder / pusher connection must be securely locked to prevent accidental release of the device from the pusher: If using the FP II, its lock mechanism (see Table 1, shown in red) must be screwed tightly. If using the OPP, check that the lock mechanism is in its ‘locked’ position.

- The Figulla® Flex II PFO Occluder is now attached to the pusher and shall be pulled into the loader while both, device and loader (distal end) are immersed in a sterile physiological NaCl-solution.

10. The following steps constitute a potential risk of introducing air into the delivery system which may lead to an air embolism. So it is important to ensure that all air-bubbles are removed from the loader and the delivery sheath by physician as described:
 - After the Figulla® Flex II PFO Occluder has been completely retracted into the loader, the loader / device assembly must be thoroughly flushed using the sterile physiological NaCl-solution to remove any air-bubbles. Please keep a saline syringe connected to the sidearm of the Y-connector.
 - Before and during connection of the loader / device assembly to the delivery sheath, it is recommended to use the slow and steady blood backstream from proximal end of the delivery sheath to remove any remaining air bubbles from the system (back bleeding).
11. Once the loader / device assembly is firmly connected to the delivery sheath, the Figulla® Flex II PFO Occluder can be advanced from the loader into the delivery sheath by pushing the pusher up to the distal end of the delivery sheath (positioned in the left atrium).
12. Once in place, the left atrial (LA) disc and the waist of the Figulla® Flex II PFO Occluder are opened by carefully advancing the pusher under continuous x-ray control. After the LA disc has been successfully deployed in the left atrium, the device is slightly pulling against the atrial septum. An elastic resistance with pulse synchronized movements can be observed. TEE, TTE or ICE imaging should be used to confirm proper positioning of LA disc at this time.
13. Once confirmed, the proximal disc of the Figulla® Flex II PFO Occluder is opened in the right atrium, by holding the pusher in place and gently retracting the delivery sheath until the device has been completely deployed.
14. Correct positioning of the Figulla® Flex II PFO Occluder must be ensured by injecting contrast agent through the delivery sheath which should also be used to appropriately visualize any residual shunt or a sub-optimally fitting product. Ultrasound doppler or bubble imaging should be used to verify occlusion at the same time.
15. A gentle pull-and-push maneuver (tug test) can be considered to verify the stable positioning of the device.
16. If the Figulla® Flex II PFO Occluder does not adequately fit upon deployment, a correction of its position must be considered (see Section 8.3: Notes for Troubleshooting). If repositioning of the device cannot be achieved, or a significant shunt remains, the Figulla® Flex II PFO Occluder should be withdrawn back into the delivery sheath, removed and discarded.
17. If the optimal fit of the Figulla® Flex II PFO Occluder has been established and confirmed by using TEE, TTE or ICE, the pusher can be safely disconnected (see Table 2, steps 17a and b). During its release from the pusher, the physician should hold the device in place with the aid of the ODS sheath. The distal tip of this sheath should be kept close to the proximal end of the device to ensure an easy retrieval in case of device dislocation.

Table 2: Releasing the Device from the Pusher

Steps	Flex – Pusher II (FP II)	Occlutech® Pistol Pusher (OPP)
<p>17a. Unlocking the Device-Pusher-connection</p>	<p>Turn the lock mechanism (shown in red) of FP II to the ‘unlock’ position.</p> 	<p>Move the unlock button to the handle of the OPP to its ‘unlocked’ position.</p> 

Steps	Flex – Pusher II (FP II)	Occlutech® Pistol Pusher (OPP)
<p>17b. Open the jaws and disconnect / release the Figulla® Flex II PFO Occluder</p>	<p>Push the handle at the proximal FP II end forward.</p> 	<p>Slowly pull the trigger mechanism located in the handle at the proximal OPP end.</p> 

Note: The physician should only disconnect the Figulla® Flex II PFO Occluder from the pusher once proper positioning of the device has been confirmed using echocardiographic imaging.

18. Further bubble, doppler and/or x-ray analysis should be undertaken to verify and document the final device deployment
19. Now, slowly retract the pusher into the ODS sheath, entirely remove both and discard them (see Section 11: Disposal).

8.3 Notes for Troubleshooting

Incorrectly positioned device

If the Figulla® Flex II PFO Occluder is not properly positioned after opening both discs, a correction of the position is required. To do so, the device must be completely withdrawn into the delivery sheath. Thereafter, another attempt to place and position the device can be made. If positioning of the device cannot be improved, the Figulla® Flex II PFO Occluder should be withdrawn into the delivery sheath, removed and discarded.

Misconfigured device

If the Figulla® Flex II PFO Occluder does not develop its intended shape during positioning, the device must be withdrawn completely into the delivery sheath by retracting the pusher. The Figulla® Flex II PFO Occluder should be removed and discarded.

8.4 Report of Serious Adverse Events

All serious adverse events which are observed in relation to any Occlutech® devices have to be reported to the manufacturer and the corresponding national authority by the physician or a delegated person (Regulation 2017/745 on Medical Devices, Chapter III of Annex I).

8.5 Recommended Follow-up Examinations

1st day after implantation

- X-ray screening for status check
- TEE, TTE or ICE (recommended; preclusion of left atrial thrombus or thrombus adhering to the device, preclusion of residual shunt)
- Electrocardiography (ECG)
- Laboratory values

30th day after implantation

- TEE, TTE or ICE (preclusion of left atrial thrombus or thrombus adhering to the device, preclusion of residual shunt)
- Clinical-neurological examination
- ECG for rhythm documentation.

60th day after implantation

- TEE, TTE or ICE (preclusion of left atrial thrombus or thrombus adhering to the device, preclusion of residual shunt)

180th day after implantation

- TEE, TTE or ICE (recommended; preclusion of left atrial thrombus or thrombus adhering to the device, preclusion of residual shunt)
- ECG

360th day after implantation

- TEE, TTE or ICE (recommended; preclusion of left atrial thrombus or thrombus adhering to the device, preclusion of residual shunt)
- ECG.

8.6 Recommended Medication after Implantation

- Standard endocarditis prophylactic therapy should be administered for a minimum of 6 months' post implantation.
- Anti-platelet / anti-coagulant therapy is indicated regarding patient conditions. 75 mg Clopidogrel for 3 months and 100 mg acetylsalicylic acid for 6 months are indicated.

9. DEVICE SIZES AND RECOMMENDED DELIVERY SYSTEMS

Compatible systems:

- Pusher: Flex – Pusher II or Occlutech® Pistol Pusher
- Delivery Set: Occlutech® Delivery Set with delivery sheath, dilator, loader and Y-hemostasis valve

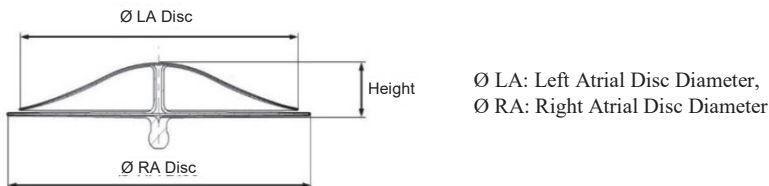
Figure 2: Representative picture of Figulla® Flex II PFO Occluder

Table 3: Device Sizes and Recommended Delivery Systems

Figulla® Flex II PFO Occluder

LA Single Layer

Figulla® Flex II PFO Occluder Article no.	Occlutech® Delivery Set Size [F]	Flex – Pusher II Article no.	Occlutech® Pistol Pusher Article no.	Ø LA Disc [mm]	Ø RA Disc [mm]	PFO Defect Size (D) [mm]
18PFO25S	9 F	51FP120 (yellow)	55PP165 (yellow)	23	25	8 < D ≤ 13

LA Double Layer

Figulla® Flex II PFO Occluder Article no.	Occlutech® Delivery Set Size [F]	Item no. Flex – Pusher II	Occlutech® Pistol Pusher Article no.	Ø LA Disc [mm]	Ø RA Disc [mm]	PFO Defect Size (D) [mm]
19PFO18D	7 F	51FP100 (light blue)	55PP125 (light blue)	16	18	D ≤ 8
19PFO25D	9 F	51FP120 (yellow)	55PP165 (yellow)	23	25	8 < D ≤ 13
19PFO30D	9 F	51FP120 (yellow)	55PP165 (yellow)	27	30	13 < D ≤ 15
19PFO35D	11 F	51FP160 (blue)	55PP210 (blue)	31	35	15 < D ≤ 18

10. STORAGE CONDITIONS

Temperature: 5 to 30 °C

11. DISPOSAL

All Occlutech® devices have to be disposed of properly according to the valid regulations, guidelines or instructions on waste disposal given by the individual clinic center.

Notes: The empty outer and sterile packaging and IFUs can be separately put in the corresponding collection containers for recycling purposes.

12. LABELLING

	CAUTION		MANUFACTURER ADDRESS
	DO NOT USE IF PACKAGE IS DAMAGED.		MANUFACTURING DATE
	DO NOT RESTERILIZE.		CONSULT INSTRUCTION FOR USE
	SINGLE USE ONLY.		SERIAL NUMBER (UNIQUE PRODUCT ID-NUMBER)
	STERILIZED WITH ETHYLENE OXIDE.		ITEM NUMBER / ORDER NUMBER
	CE MARK. PRODUCT MEETS THE REQUIREMENTS ACCORDING TO 93/42/EEC		STORE DRY & AWAY FROM SUNLIGHT
	USE BY DATE. / EXPIRE DATE.		TEMPERATURE LIMIT FOR (LONG TERM) STORAGE
	DIAMETER [mm]	Quantity 1	QUANTITY

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**Occlutech GmbH